EN: Quality-Update *eBioo* 1.85 now available

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EN: Quality-Update *eBioo* 1.85 – Improved Functionalities: DVD-Player, Trend-Screen, Assessment, Reward-Ratio and Threshold-Evaluation, new Quick-Lookup Menu-Options for Report-, Screenshot- and Client-Data-Localisation;

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eBioo is best performed with **Neurobit Optima** hardware.

EN: Quality-Update *eBioo* 1.85 now available – several improved Functionalities:

After hints and suggestions of clients that are using *eBioo* the following *eBioo* functionalities have been improved:

DVD-Player: The DVD-Player in *eBioo* now is supported by VLC-interface.

The DVD-Player was introduced in *eBioo* version 1.80; The DVD-player in that version was based on Windows API functions. Now *eBioo* 1.85 has an additional DVD-Player included, that is based on an interface that is provided by VideoLAN, see www.VideoLAN.org, that works on more Windows versions.

Trend-Screen: The layout of the trend screen option selection and the Amplitude-Scaling has been improved. The trend screen provides two different trend types:
a) Spectrum Amplitude values and b) Training-Frequency-Window values.

As spectrum Amplitudes the amplitude type (Band-Amplitude, Amplitude, relative Amplitude and Coherence can be charted). The Training-Window values can be charted as real value and as score. The score is a value with positive sign (the value is greater than the threshold) and negative sign (value is less than the threshold value). New: The amplitude scaling in the trend screen is amended to show the real value of the according training function selected of the appropriate training window (Only if all training windows set active (reward or inhibit) have the same function selected, so that all training windows have the same unit. If the active training windows have different functions, the Y-Skala is sketched and normalized as %-value (0 up to 100%) where 100% is the maximum of the appropriate function.

Assessment: The assessment now has more help texts popping up while performing the assessment, so that if someone is performing the assessment the first time, now is getting more help texts. Once the trainer is used to the assessment procedure he/she does no longer need these explanation texts.

Quick-Lookup **Menu-Options:** The main menu on top now has several new menu items to allow quick access to the reports, screen shots and client data.

The following new main menu options have been added in version 1.85:

Client -	Login
	Lookup Client Reports
	Lookup Client Screenshots
Assessment	Stop Assessment
	Lookup Client Assessment Reports
Session Design	Load Common Design (subDir: predefined_Trainings)
	Load Client Design (from Client specific Folder)
	Save Design as Common Design
	Save Design as Client design
Session	Lookup Client Session Reports
Screenshots	Lookup Common Screenshots
	Lookup Client Screenshots

The file manager form now contains the name of the directory and sub-directory to give an orientation. Up to the previous *eBioo* versions the file manager to load the files opened the directory that was last visited, and did not mention the full directory name. From version 1.85 on the file manager opens in the appropriate common or client folder.

Reward-Ratio and Threshold-Evaluation:

Some clients using *eBioo* observed that the global reward ratio was way above the targeted reward ratio. That effect could happen, in case that the „Reward Threshold Update Rate“ under „Options – Program - Reward Threshold Update“ was configured to slow (i.e. Slower than every 30 seconds). In that case configure the reward ratio update to be quicker.
The check the pace of the reward ratio update, see: „Trend Screen“ and select: „Training Window“

with „Value“ and „Threshold“ checked/activated. In that case, both, the Amplitude value together with the threshold of that particular training window is sketched. Do some quick variation to the Amplitude (e.g. using the function generator).

The *eBioo* STARTER version 1.85 can be downloaded on my internet site www.Soft-dynamics.com

System Requirements:

Currently there are 3 different Windows operating systems in use (Win 7, 8 and 10) on 2 processor types (32 and 64 Bit), countless monitor resolutions, computers with 1 or 2 monitors, 3 EEG amplifiers types and now 2 DVD APIs (Win und VLC) available. That alone are already $3 \times 2 \times X \times 2 \times 3 \times 2 = 72 \times X$ different possible combinations. Thank You to all Users of *eBioo* that report any unexpected behaviour or error on a particular system.

Prospect: Video codec with advanced set of video formats:

Occasionally, individual videos can not be displayed and a raster graphic appears. If necessary, a replacement of the video codec is required. Users who find non-displayable video formats can contact soft-dynamics to schedule a selection and replacement of the video codec.

***eBioo* 2.0 coming (!?) :**

As next generation *eBioo* software is considered to have

- T-Score Training (= estimated Z-Score-Training)
- High resolution T-Score Brain-Maps
- Target-Training (configurable T-Score-Values with Target Z-score not like 0)

Users that are interested to do test runs with *eBioo* 2.0 to test the T-Score-Training and exchange their experience are asked to contact Soft-dynamics.

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